




UA BRACES

FIT INSTRUCTIONS

UA BRACES STRAPPED
UA BRACES STRAPLESS
UA HOOPS BRACES
UA SOCCER BRACES

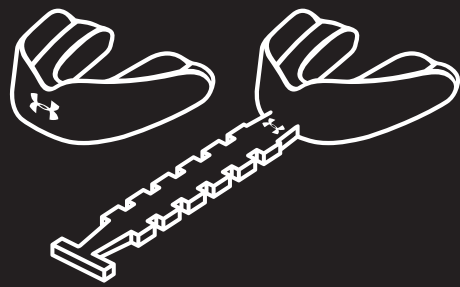

CARE INSTRUCTIONS

- UA Mouthwear Freshener spray is recommended after each use to keep your mouthguard fresh and clean.
- Protect your mouthguard by storing it in the UA Mouthwear case.



Your strapped mouthguard can be easily converted to a Trim-Tab.

1. Fit your mouthguard according to the fit instructions.
2. Securely hold the mouthguard, and using scissors carefully cut off the strap just above the UA logo.

NOTE: Scissors are sharp. Use caution when handling.

STOVE-TOP

MICROWAVE

READ ALL INSTRUCTIONS PRIOR TO FITTING YOUR MOUTHGUARD.

What you need for a stove-top fitting:

- Pot for boiling water 4" deep
- Slotted spoon
- 9" x 9" pan of cold water
- Mirror
- Timer

What you need for a microwave fitting:

- Microwave oven (1100 watts +)
- Wet paper towel
- Microwave-safe dish
- Mirror
- 9" x 9" pan of cold water
- Timer



Bring pot with 4 inches water to a boil and then remove from heat source ** and allow water to cool for 30 seconds.



Wrap mouthguard in wet paper towel and place on microwave safe dish.



Submerge mouthguard in hot water for 30 seconds.
Strapless - carefully remove mouthguard from hot water with slotted spoon.
Strapped - do not submerge the strap.



Microwave mouthguard for 45 seconds on high then remove from microwave. **



Immediately submerge mouthguard in pan of cold water for no more than 5 seconds. Do not allow sides to fold over as the material will "glue together" making it difficult to get an optimal fit.



Immediately unwrap mouthguard and submerge in pan of cold water for no more than 5 seconds. Do not allow sides to fold over as the material will "glue together" making it difficult to get an optimal fit.



Gently glide your thumbs inside the molar channel to ensure there is ample space for the mouthguard to fit around your teeth. Using a mirror, carefully place the mouthguard in your mouth, lining up the "notch" in the center of the mouthguard with the center of your front teeth



Gently glide your thumbs inside the molar channel to ensure there is ample space for the mouthguard to fit around your teeth. Using a mirror, carefully place the mouthguard in your mouth, lining up the "notch" in the center of the mouthguard with the center of your front teeth.



To ensure proper fit, bite down gently and naturally on mouthguard for 1 minute. Avoid excessively clenching molars. **DO NOT SUCK ON MOUTHGUARD OR PRESS ON SIDES OF MOUTHGUARD.**



To ensure proper fit, bite down gently and naturally on mouthguard for 1 minute. Avoid excessively clenching molars. **DO NOT SUCK ON MOUTHGUARD OR PRESS ON SIDES OF MOUTHGUARD.**



After fitting process is complete, remove mouthguard from mouth and lay completely flat in 9" x 9" pan of water for 5 minutes to allow the mouthguard to cure/harden.



After fitting process is complete, remove mouthguard from mouth and lay completely flat in 9" x 9" pan of water for 5 minutes to allow the mouthguard to cure/harden.

RE-FITTING


If re-fitting is desired, repeat steps 1-6.

RE-FITTING

If re-fitting is desired, repeat steps 1-6.

UA Braces is designed to be worn on the upper teeth and must be fitted as instructed to function properly. A damaged, altered or improperly fitted mouthguard will not provide the same protection.

**** CAUTION: BOILING WATER AND ITEMS THAT HAVE BEEN MICROWAVED WILL BE HOT AND CAN CAUSE INJURY.**

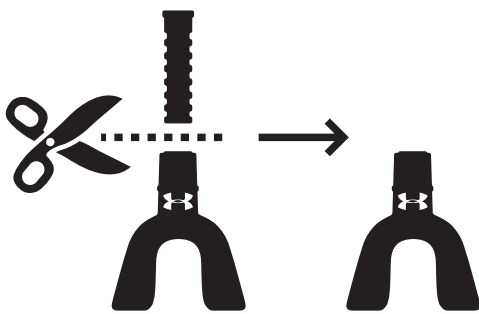

DENTAL LIMITED WARRANTY

CERTIFICATE OF LIMITED WARRANTY AND DISCLAIMER

WARRANTY: Bite Tech, Inc. (Bite Tech) warrants this mouthguard to be free from defects in material and workmanship which could cause damage or injury to the sound, natural teeth of the purchaser herein when properly fitted and used for a period of one (1) year from the date of purchase.

LIMITATION OF LIABILITY: This limited warranty applies only to injuries to the purchaser's teeth that are free from any pre-existing natural or unnatural defect, including prior dental work. This limited warranty only applies if the purchaser suffered a dental injury in a supervised athletic contest or coach-supervised training session not otherwise excluded. No warranty is provided for any damage or injury incurred during or in connection with any martial arts, mixed martial arts, boxing or similar fighting or combat sport or activity. Bite Tech's obligation under this warranty shall be limited to the actual costs incurred by the user for necessary dental treatment by an appropriately licensed dental professional up to a maximum liability of \$500 per injured or replaced tooth with a maximum liability per incident of \$32,000. Bite Tech's obligation shall be reduced by any insurance coverage available to the user and applicable to the treatment of the injury claim. This limited warranty does not provide payment or indemnity for orthodontic treatments or preparation of a tooth to be used as an anchor for a bridge. The remedies set forth herein are exclusive and the total liability of Bite Tech with respect to use of this mouthguard, whether based on contract, warranty, negligence, indemnification, strict liability or otherwise. In no event shall Bite Tech be liable for consequential, incidental or special damages.

CONDITIONS: This limited warranty applies only to injuries to the purchaser's teeth that are free from any preexisting natural or unnatural defect, including prior dental work. This limited warranty

only applies if the purchaser suffered a dental injury in a supervised athletic contest or coach-supervised training session. Bite Tech shall have no responsibility or liability hereunder, unless:

1. the mouthguard was fitted in accordance with the printed instructions that accompany the mouthguard; and
2. the mouthguard was not damaged or altered in any way, whether by chewing on the mouthguard or by other action to alter its configuration or chemistry (other than by the prescribed fitting), except as occurs by natural use; and
3. a complete, written notice of the injury and damage, together with a copy of the receipt of purchase, is received by Bite Tech within thirty (30) days after the occurrence of the injury; and
4. the user provides a written statement from an appropriately licensed dentist providing necessary treatment within thirty (30) days after the date of injury that substantiates the injury, damage and treatment provided, together with identification of insurance coverage and insurance payments; and
5. the user provides a statement by a supervising coach or participating game official that the mouthguard was in proper use at the time of injury; and
6. the applicable mouthguard is returned to Bite Tech with the written notice of injury. The mouthguard shall become the property of Bite Tech upon receipt of return.

NO MOUTHGUARD CAN PREVENT ALL INJURIES OR ALL DENTAL DAMAGE AND NO SUCH WARRANTY IS BEING MADE. THIS WARRANTY IS MADE IN LIEU OF ALL OTHER WARRANTIES EXPRESS OR IMPLIED, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTIES FOR A PARTICULAR PURPOSE AND ANY IMPLIED WARRANTY ARISING OUT OF A COURSE OF A DEALING, CUSTOM, USAGE OR TRADE, EXCEPT AS IS OTHERWISE EXPRESSLY PROVIDED HEREIN, BITE TECH DISCLAIMS ALL LIABILITY FOR LOSS, INJURY OR DAMAGE BASED UPON NEGLIGENT DESIGN OR DEFECTS IN MATERIALS AND WORKMANSHIP FOR THE WITHIN MOUTHGUARD. Keep this dental warranty, disclaimer and store receipt in your records. In case of injury, mail mouthguard, store receipt, the notice of injury and accompanying statements to Bite Tech, Inc. This product is protected by one or more U.S. and international patents and pending patent applications. Please visit www.ArmourBite.com for more details.